


HOVERSLING® USER MANUAL

Part Identification – HoverSling

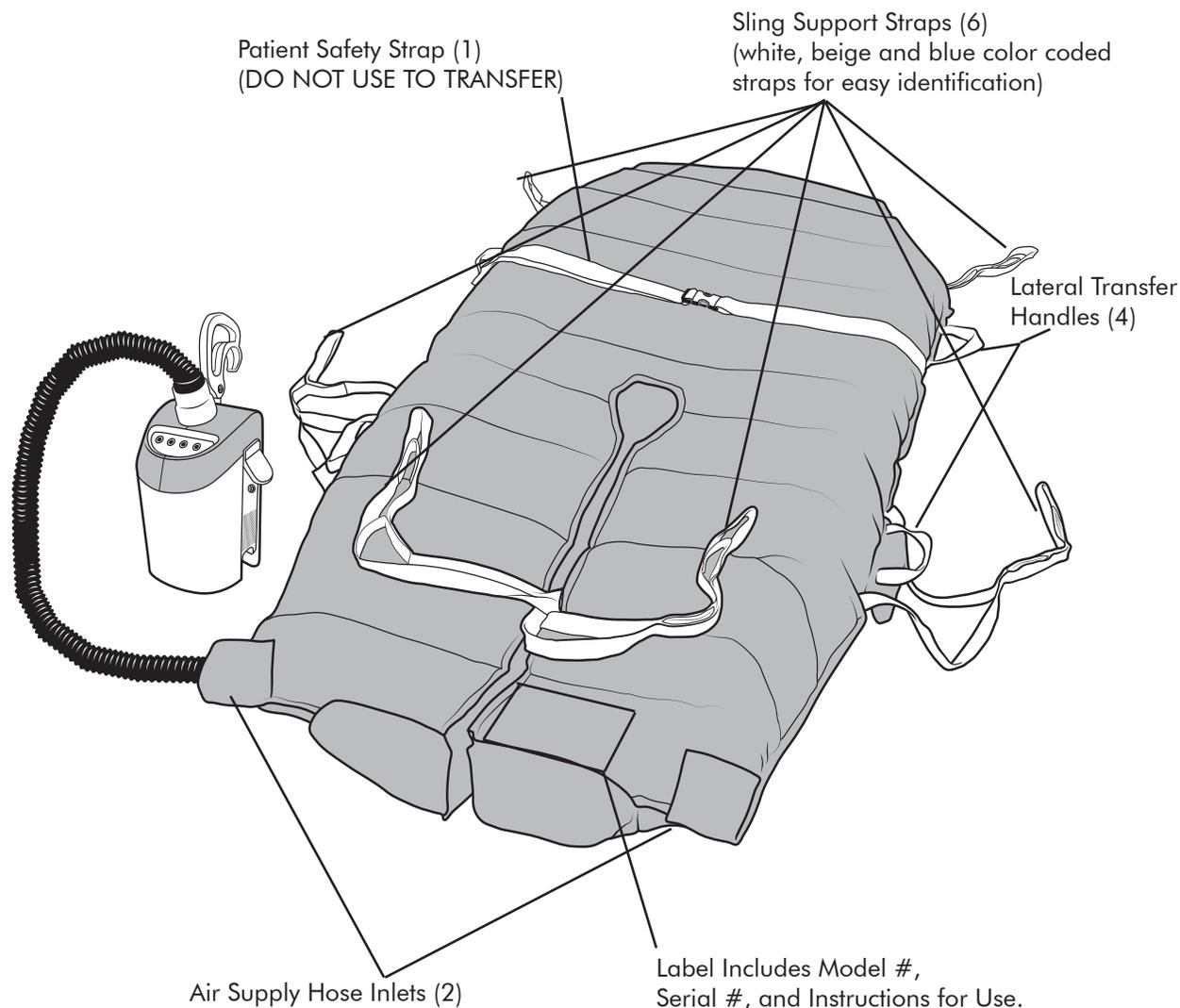

Before each use, check to ensure:

- | | | |
|------------------------|-------------------------------|---------------------|
| ✓ Safety straps intact | ✓ Handles intact | ✓ No tears or holes |
| ✓ Sling straps intact | ✓ Do Not Launder label intact | ✓ No loose threads |




HOVERSLING® USER MANUAL

Instructions for Use as a Transfer Mattress

1. Ensure sling support straps are stowed in sleeves and the four (4) snaps located at the center leg and foot sections are connected.
2. Patient should be in a horizontal position for transfer/repositioning.

Regardless of ease of patient movement, for safety, always use a minimum of two caregivers for the transfer.

3. Place the HoverSling underneath patient using log-rolling technique and attach patient safety straps. Whatever the patient is lying on to keep the bed mattress clean can be placed on top of the HoverSling.
4. Plug the Air Supply power cord into an electrical outlet.
5. Insert the Air Supply hose nozzle into mattress in one of the two hose inlets located parallel to the foot end, and snap in place.
6. Be sure transfer surfaces are as close as possible and brake the wheels of the bed/stretcher.
7. If possible, transfer from a higher surface to a lower surface.
8. Turn on Air Supply by choosing the appropriate speed:

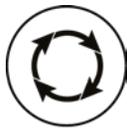

HOVERMATT®

Use for sizes:
28" & 34"

Or



HOVERMATT®

Use for sizes:
39" & 50"

9. Grasp transfer handles and pull patient on an angle, either head first or feet first, until patient is in desired position.
10. Ensure the patient is centered on the receiving equipment prior to deflation, especially if the width of the equipment receiving the patient is less than the width of the HoverSling.
11. Press Standby to deflate HoverSling and employ the bed/stretcher rails.

NEVER LEAVE PATIENT UNATTENDED ON AN INFLATED HOVERSLING.



T: 1300 106 003

F: 07 3036 6625

E: info@hmsmedical.com.au