

Staff Training Guidelines for the HoverJack

Instructions for Use:

1. Place HoverJack air patient lift on floor next to the patient, making sure the chamber with Valve #4 is on the top and the chamber with Valve #1 is against the floor.

For safety, always use a minimum of two caregivers when operating the HoverJack air patient lift.

2. Make certain that all four red-capped deflation valves are capped tightly to maintain inflation.
3. Log roll patient onto the deflated HoverJack air patient lift and position patient with feet at the valve end where indicated.
4. The patient can be placed on top of the HoverJack air patient lift using the HoverMatt Air Transfer System. (See HoverMatt instructions). If the HoverMatt is used, make certain that the HoverMatt and patient are properly centred on the HoverJack air patient lift. Always deflate the HoverMatt air transfer system prior to inflating the HoverJack air patient lift. Using buckles, secure patient safety straps around the patient.
5. Plug HoverTech International Air Supply power cord into an electrical outlet.
6. Hold hose against inlet Valve #1 of HoverJack air patient lift.
7. Turn on Air Supply to the highest inflation level to begin inflation with Valve #1.



HOVERJACK®
Use for sizes: 32 & 39

8. When fully inflated, remove hose. Valve will automatically close, keeping chamber inflated.
9. Using the same process, move to Valve #2, Valve #3 and Valve #4 in exact succession.
10. Turn off Air Supply by pressing standby button, and cap valves.
11. Transfer from HoverJack air patient lift onto adjacent surface using the HoverMatt air transfer system. Transfer without the HoverMatt air transfer system may cause injury.
12. If it is necessary to lower patient down to the floor, release air by opening the uppermost red deflate Valve #4. When chamber #4 is fully deflated, move in succession downward to fully deflate HoverJack air patient lift. Caution: DO NOT RELEASE ALL CHAMBERS ALL AT ONCE.

NEVER LEAVE PATIENT UNATTENDED ON INFLATED HOVERJACK AIR PATIENT LIFT.

