


---

# HT-Roller™ Lateral Turning Device

## Part Identification

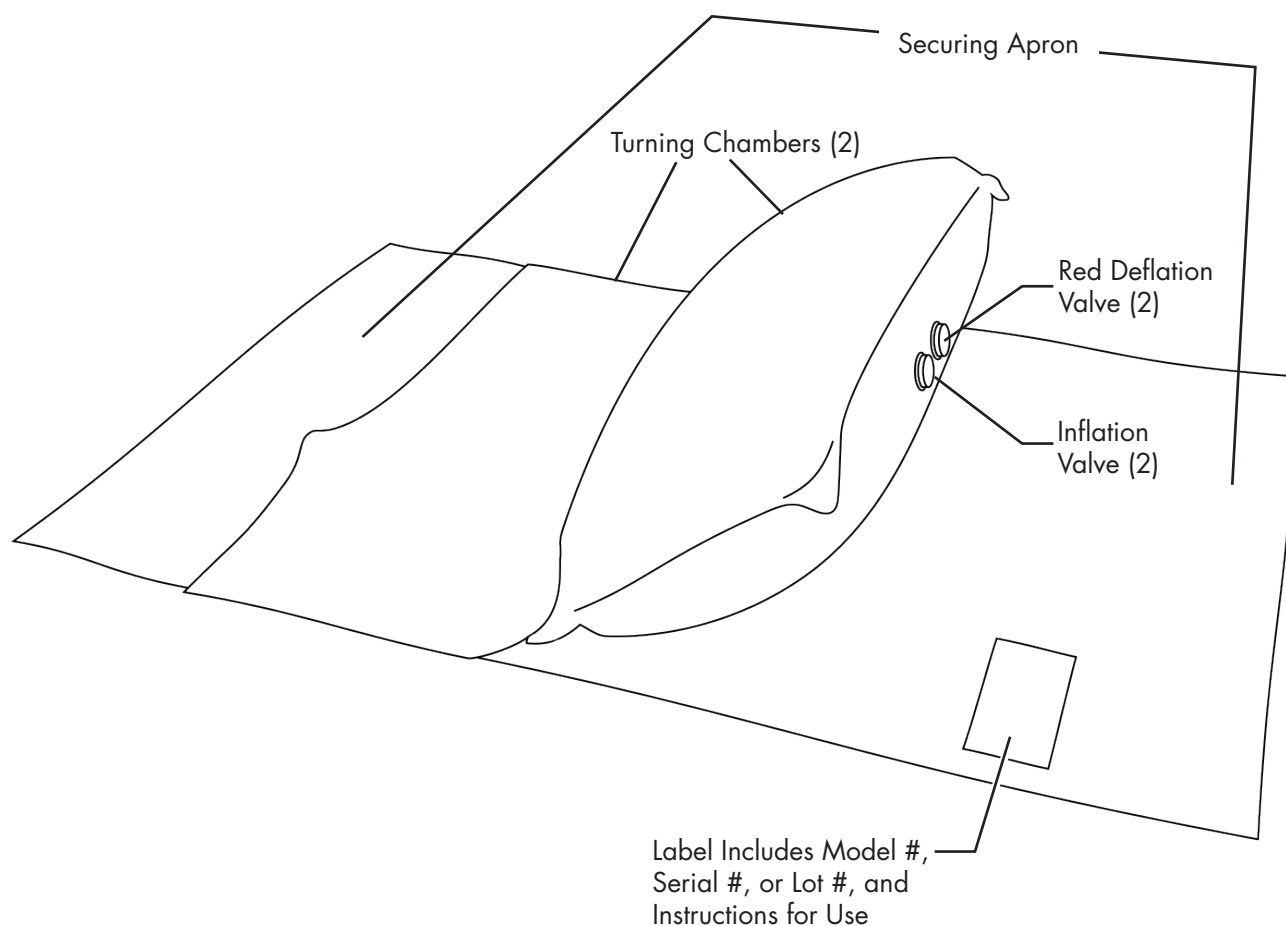

T: 1300 106 003  
E: [info@hmsmedical.com.au](mailto:info@hmsmedical.com.au)



---

# HT-Roller™ Lateral Turning Device

## Instructions for Use

1. Ensure that the red valves are capped tightly before beginning inflation.
2. Place the deflated HT-Roller on the bed mattress. If the patient is already on the mattress, place the device under the patient by using a log-rolling technique. Ensure all side rails are up. Whatever the patient is lying on to keep the bed mattress clean can be placed on top of the HT-Roller.
3. Tuck the securing apron of the HT-Roller under the bed mattress to keep in place during use.
4. Locate the inflation valve that is positioned on the OPPOSITE side of the turn direction and place the Air Supply hose over the valve. Press the ADJUSTABLE button on the HoverTech HTAIR1200 keypad to initiate air flow. If necessary, press this button up to 3 more times to increase air flow and rate of inflation.



### **ADJUSTABLE**

Use for HT-Wedge and  
HoverApps

5. When the turn is complete, remove hose from valve and press the STANDBY button on the Air Supply keypad to stop air flow.  
**Note: Chamber may not need to be fully inflated to achieve turn.**
6. To deflate the HT-Roller, remove the red cap from the deflation valve. This will quickly deflate the positioning device. **Make sure the patient is attended and stabilized during deflation.**



T: 1300 106 003  
E: [info@hmsmedical.com.au](mailto:info@hmsmedical.com.au)  
[www.hmsmedical.com.au](http://www.hmsmedical.com.au)

