



NEO-SNUG

Stress free infant positioning

User instructions

The Neo-Snug is designed to assist the nurse in providing a safe and secure positioning “nest” for the Neonate.

1. Remove the Neo-Snug from its bag and position it in the crib with the closed padded foot rest towards the bottom of the crib and the wraps on one side, as shown in the image.
2. The Neonate can be positioned in a number of ways within the Neo-Snug, on the side, prone or supine. When placing the Neonate in the Neo-Snug ensure that the feet are tucked in against the padded foot rest and place the hands close to the face. It is critical that the flexion is correct. Please refer to the hospital or therapist for further assistance.
3. The two wraps now need placing over the Neonate. Place the top wrap over first as illustrated. Finish by tucking each wrap under the Neo-snug gently but firmly on the other side with the hook and loop fastener. Adjust according to comfort for the Neonate.
4. The bottom wrap must be checked for the right positioning of the legs. Finally check all connecting tubes or lines for security and positioning.
5. The Neo-Snug can come with or without Security Clips for tubing. It will help to hold the tubes when the baby is out of the incubator.
6. An extra wrap (without attachment to the base) is provided if the situation requires this.

N.B. the flexibility of the Neo-Snug means that the positioning of the wraps can be changed according to the comfort of the Neonate or to assist the nurse in monitoring.



Neo-Snug (Disposable) - re-ordering

- NSG001C Neo-Snug size 01 - < 1100g with clips
NSG002C Neo-Snug size 02 - 1100g to 1800g with clips

Conlett Technology Inc. 

Distributed by: Conlett Pty Ltd

Unit 25/17 Cairns St
Loganholme
QLD 4129
Australia

E: contact@conlett.com
W: www.conlett.com