

# Instructions for use



# Bure<sup>TM</sup> Walker

**GATE**

REHAB DEVELOPMENT AB

Bure Standard MANUAL 56-310  
Bure S MANUAL 56-300


## ***Dear User,***

*Congratulations on obtaining a Bure Walker! These instructions provide important information on how to use the walker and how to maintain it.*

*All of us here at GATE Rehab Development hope you enjoy using your new product. If you have any comments, questions or points of view, please contact us on +46 (0)371 318 00.*

## **Raising and lowering the frame**

Adjust the height by undoing the screws on each side (two pcs). Raise or lower to the desired height and then lock the screws (2 pcs).

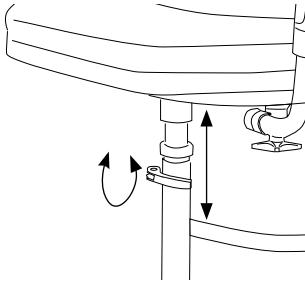


## Adjusting the handles

To adjust handles to the desired position; loosen screw, adjust handle, tighten screw.



## Left and right adjustment of armrests

The armrests can be adjusted sideways for optimal support; loosen screw, adjust armrest, tighten screw.

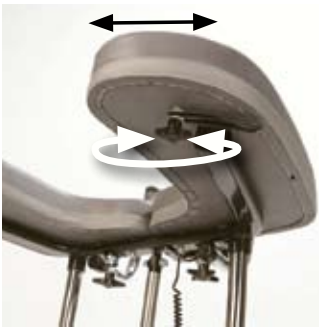

## Locking castors

Depress the foot brake to set it and lock the castor.



## Unlocking castors

Press down on the protruding part of the brake to release it.



# Maintenance

Recommended at least once per year.

Check that	Action
<b>General</b> The walker feels firm/free from play.  The walker does not rattle during manoeuvres.  The walker is level and all of its castors are in contact with the floor.  The walker is not dirty.	Clean with lukewarm soap solution or alcohol-based cleaning agent (no petroleum products).
<b>Armrests</b> Armrests are intact and clean. Armrest widening is functional.	Clean; replace if damaged. Fit lock screws/plates as required.
<b>Handle</b> Grips are not damaged/dirty.  Handle adjustment is functional.	Clean with lukewarm soap solution (no petroleum products). Replace handle grips. Fit lock screws/plates as required.
<b>Height adjustment</b> Raising and lowering are functional.  The walker feels stable at maximum settings.  There is no play between the vertical frame and bottom frame.  Height adjustment clamps lock.	Fit new guide sleeves (in frame) or end plugs (in chrome tubes) as required.  Tighten all lock bolts.  Replace clamps.
<b>Frame parts</b> There is no mechanical damage.  There are no scratches. End plugs/lower frame fitted.	If damage is present, contact GATE's customer services. Touch up as necessary. Fit new end plugs.
<b>Castors/brakes</b> Castors roll easily/tread not damaged.  The castors are firmly fastened to the lower frame.  Castor brakes functional on all castors.  Handbrake functional.	Clean or replace castors. NOTE! Castors are always fixed to the lower frame with thread locker or lock nuts. Do not dismantle the castors; replace the whole castor assembly instead.  Tighten the castor bolt and fit locking nut or use thread locker (depends on model).  Replace castor assembly.  Adjust the brake or fit new castors.

## General maintenance advice

- Clean the walker and its armrests clean as necessary. Wipe using a soap solution/surface disinfectant.
- The product can be washed in a pressure washer/machine at 60°C for ten minutes. To increase lifetime, avoid aiming high pressure water jets at bearings.
- Replace damaged parts immediately; must be done by experienced personnel.
- The product is only intended for indoor use.

## Technical specifications

Dimensions apply to walkers with 125 mm castors.

Standard Manual	S Manual
<b>Part No.</b> 56-310	<b>Part No.</b> 56-300
<b>Width</b> 78 cm	<b>Width</b> 66 cm
<b>Max user weight</b> 150 kg	<b>Max user weight</b> 150 kg
<b>Length</b> 80 cm	<b>Length</b> 79 cm
<b>Height</b> 91–133 cm	<b>Height</b> 91–133 cm
<b>Armrest/internal dimension</b> 25–50 cm	<b>Armrest/internal dimension</b> 30–45 cm

**CE** CE marked according to MDD 93/42/EEC.  
Tested and approved according to EN-11199-3/2005



YOUR HEALTHCARE PARTNER

Would you like to know more?

Contact us at:  
Tel: +46 371 31800  
Gate Rehab Development AB  
Industrigatan 2 | SE-330 21 Reftele  
info@gaterd.se | www.gaterd.se



GATE COLLECTION