



# Bari-suit®

## User Manual







Benmor Medical has been supporting bariatric care since 1996 and was the first company in the UK specialising in bariatric patient handling equipment.

As pioneers in the sales and rental market for bariatric equipment, Benmor Medical introduced total bariatric sales and rental solutions, allowing our customers to rent or purchase a bed, chair, hoist, commode and wheelchair as one package.

We continually strive to drive innovation and quality within the bariatric market and have established a firm reputation for excellent customer support and class leading products.

Benmor Medical supplies and rents a wide range of products which enable nurses and carers to provide the best quality care to patients.

Our extensive product portfolio includes: -

- Bariatric Beds
- Ultra Low Beds
- Community Beds
- Mattresses
- Hoists
- Riser Recliner Chairs
- Static Chairs
- Commodes
- Walking Aids
- Bariatric Wheelchairs

Our well established reputation in the UK bariatric sector has been achieved by providing excellent 24/7 customer service, supported by innovative products.

For further information, please visit our website: [www.benmormedical.co.uk](http://www.benmormedical.co.uk)

There are 3 main parts to the bari-suit and an additional 2 attachments that can be added/removed to alter the shape of the suit.

1) Trousers



2) Body



Front with all parts attached.



Front with all parts detached.



Back with all parts attached.



Back with all parts detached.

3) Arms with optional attachments to bari-suit to alter to a 'pear' shape.





Arms with optional attachments attached (back)



Arms with optional attachments attached (front)



The following step by step instructions are guidelines for wearing the bari-suit.

1. To begin with remove any footwear and step into the trousers and attach the elastic braces.



Adjust length of trousers if necessary by turning up the bottom part of the legs.



2. Once the trousers have been fitted apply the arms section, with or without the optional attachments.



If the optional attachments on the arms section have been added tuck them into the trouser legs.

3. Apply the body section over the top of the arms section to complete the suit. It should look like the images below.

