



SYST'AM® P361C/VISCOELASTIC CUSHION

MOULDED CUSHION MADE OF VISCOELASTIC FOAM WITH MEMORY EFFECT

MATERIALS

80 kg/m³



Visco foam



Removable POLYMAILLE® cover



Removable POLYMAILLE® HD cover

NF EN ISO 597 - 1 & 2

Foam maintenance:



(Do not immerse in water)

Cleaning of the cover:



Cleaning POLYMAILLE® HD cover:



INDICATIONS

Risk



P361C / VISCOFLEX® cushion can be used, when:

- 1- already pressure sore (Stage I to II according EPUAP) is present. The patient can change seating position or the pressure points are relieved through nursing care. OR
- 2- an increased to very high risk according evaluation scale (Braden, Norton, Waterlow, ...) is present. The P361C / VISCOFLEX® cushion is suitable for patients: with past history of pressure sore on contact with seating area, and/or also complete or incomplete paralysis of the trunk or lower limbs, and/or with impaired or absent sensation of the lower member, and/or with impaired frontal stability, and/or with light postural asymmetries and/or with vascular Amputation, and/or with bad general state.

The cushion can be used on different seating support without adjustment.

AVAILABLE VERSIONS



P361C / VISCOFLEX® cushion
With POLYMAILLE® cover.



P361C / VISCOFLEX® cushion
With POLYMAILLE® HD cover
(with welded seams).



P361C-S / VISCOFLEX® Soft cushion
With POLYMAILLE® cover,
for Patients < 70 kg.

DESIGNATION	ITEM CODE	 (W x L - cm / inches)	 min - MAX (kg / lbs)
P361C / VISCOFLEX® cushion with POLYMAILLE® cover	P361C36361HW	36 x 36 cm / 14 x 14"	30 - 50 kg / 66 - 110 lbs
	P361C38381HW	38 x 38 cm / 15 x 15"	30 - 90 kg / 66 - 198 lbs
	P361C40401HW	40 x 40 cm / 15,5 x 15,5"	40 - 100 kg / 88 - 220 lbs
	P361C42421HW	42 x 42 cm / 16 x 16"	40 - 100 kg / 88 - 220 lbs
	P361C42461HW	42 x 46 cm / 16 x 18"	40 - 110 kg / 88 - 242 lbs
	P361C45421HW	45 x 42 cm / 18 x 16"	40 - 110 kg / 88 - 242 lbs
	P361C45461HW	45 x 46 cm / 18 x 18"	40 - 130 kg / 88 - 286 lbs
	P361C48431HW	48 x 43 cm / 19 x 17"	40 - 140 kg / 88 - 308 lbs
	P361C51461HW	51 x 46 cm / 20 x 18"	60 - 150 kg / 132 - 330 lbs
	P361C51501HW	51 x 50 cm / 20 x 20"	60 - 160 kg / 132 - 352 lbs
P361C-S / VISCOFLEX® Soft with POLYMAILLE® cover	P361CS42421HW	42 x 42 cm / 16 x 16"	30 - 70 kg / 70 - 154 lbs
P361C / VISCOFLEX® cushion with POLYMAILLE® HD cover	P361C42421HF	42 x 42 cm / 16 x 16"	40 - 100 kg / 88 - 220 lbs
	P361C45421HF	45 x 42 cm / 18 x 16"	40 - 100 kg / 88 - 220 lbs
	P361C45461HF	45 x 46 cm / 18 x 18"	40 - 130 kg / 88 - 286 lbs
	P361C51501HF	51 x 50 cm / 20 x 20"	60 - 160 kg / 132 - 352 lbs







MOULDED VISCOELASTIC FOAM WITH MEMORY EFFECT

- Precise moulding of the body and increase of the body surface in contact with the cushion;
- Better pressures distribution: reduction of transcutaneous pressures on areas at high risk and facilitated blood circulation;
- Improved comfort and stability of the patient;
- Skin effect obtained through the moulding process: protection of the foam against external aggressions (tear, crumbling);
- With a very high density to prevent deformation and sagging effects.

SURFACE CONTAINING VENTS

- Better circulation of air to reduce the effects of maceration.
- Better accompanying of the body by the material during movement to reduce surface frictions.

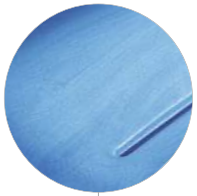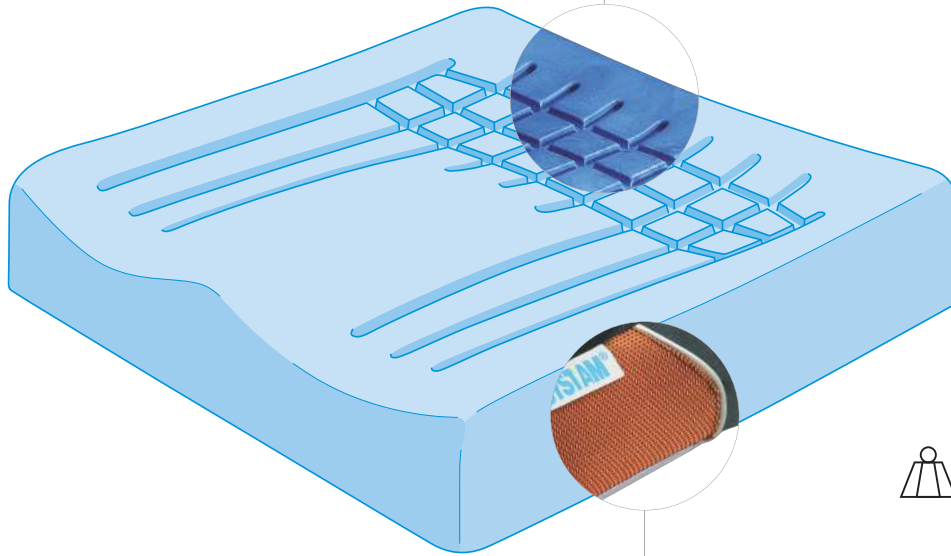

SKIN EFFECT FROM MOULDING PROCESS

- Protects the product against external aggressions.





ULTRA-LIGHT

HIGHLY BREATHABLE COVER

- Adapted for incontinence
- Reduces the maceration effects through easing the skin respiration.
- Impermeable material

ANATOMICAL SHAPE WITH RAISED LATERAL EDGES AND FRONT-BACK SLOPE

- Improves the positioning, the stability and the comfort of the patient;
- Increases the contact surface and therewith reduces the pressures.

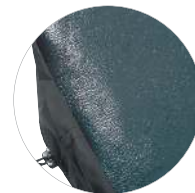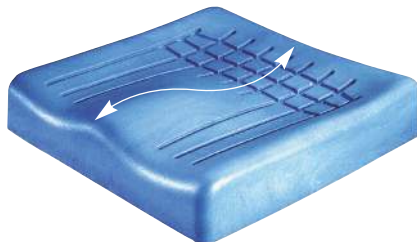

AN ANTI-SLIP BASE

- Avoiding any sliding of the cushion from the support (POLYMAILLE® only).



AVAILABLE IN
10 DIFFERENTS SIZES

Distributor:



T: 1300 106 003
Dir: 07 3117 9539
F: 07 3036 6625
E: sales@hmsmedical.com.au