



MANUFACTURER
Conlett Technology Inc
908, Building 2, 28 Pingguoyuan Lu,
Shijingshan. 10004301
Ph: 010-85295613

DISTRIBUTED BY
Conlett Pty Ltd
Unit 25, 17 Cairns St
Loganholme
QLD 4129
Australia

INSTRUCTIONS FOR APPLYING NEO-SHIELD

1. Ensure that the area around the sensor is clean and dry. This will aid with the adhesion of the Neo-Shield.

2. Remove the Neo-Shield from it's packaging.



3. Peel the Neo-Shield from the backing film.

4. Place the Neo-Shield on top of the sensor. Ensuring the sensor is central to the Neo-Shield and the edges of Neo-Shield are adhered to the patient.

5. If the positioning of the Neo-Shield is not ideal after the first attempt, it can be removed and reapplied until the ideal position is achieved.



6. Check for dislodgement at regular intervals.