



For Home & Care Facilities

The IndeeLift FTS-400 easily lifts mobility challenged individuals from the floor to a sitting or standing position.

The FTS-400 can be self-operated or operated with the assistance of a helper, safely lifting a person up to 30" from the floor to achieve a full standing position.

FTS-400 (Floor-to-Stand)


Self-Operation

After a fall, the person scoots, crawls, or rolls over to the FTS-400, slides or tilts onto the seat, pushes a button on the remote, raising them from the floor to any point that is comfortable for them to stand. The FTS-400 can also be stopped at the height of an exercise bike, bed, chair, couch, or wheelchair for a safe transfer – with the FTS-400 doing all of the heavy lifting!



Assisted Operation

A helper/caregiver easily wheels the FTS-400 to the fallen person and assists them onto the seat, if necessary. The helper pushes a button on the remote, raising the fallen person from the floor to a standing position or to a level that allows the fallen person to transfer as required.

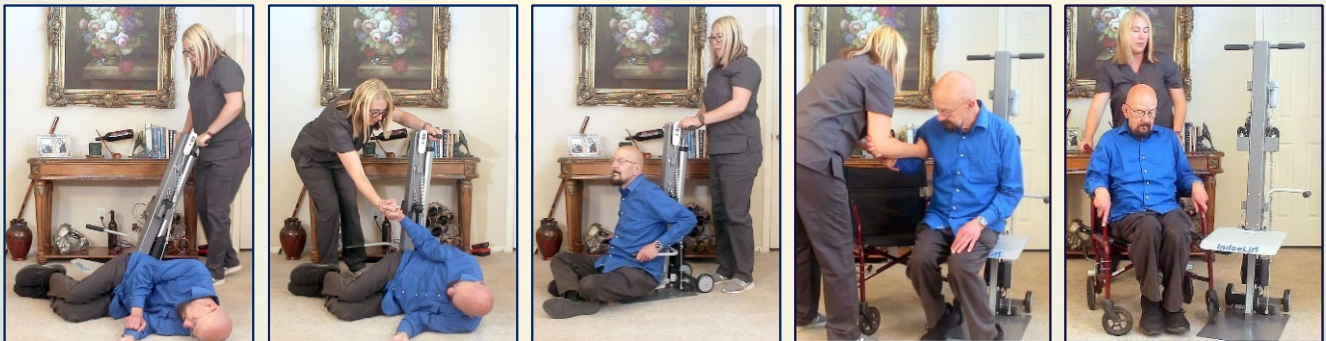

Standing Function: Reaching a Tall Cabinet



Dual Belt Assemblies for Additional Safety




Self or Assisted Operation

FTS-400 (Floor-to-Stand)

Description

The Floor-To-Stand (FTS-400) is a lift & transfer tool that enables safe patient handling by eliminating manual patient lifting in a variety of situations. The FTS-400 raises individuals to a standing position from any practical location one might sit; wheelchair, commode, couch, bed, or even the floor!

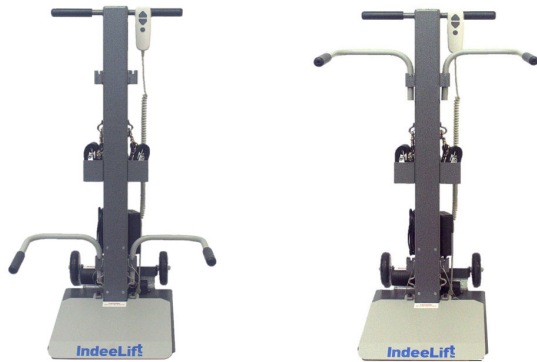


Removable Rise-Assist Handles
Shown in Lower and Upper Mounting Positions

Summary of Features

- Self or assisted operation
- Beveled seat-front for easy slide-on access
- Wired remote for convenient operation
- Swinging/Removable Rise Assist Handles for secure easy transfers
- Small footprint for tightspaces
- Powder coated for easy cleaning
- Self-contained rechargeable Lithium Ion battery & charger with 9' (2.74m) power cord

Component Identification

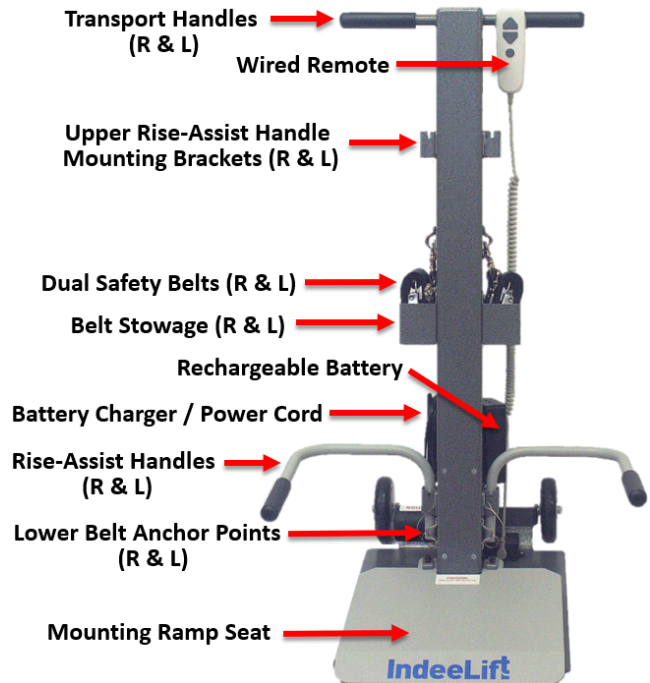

Physical Specifications

Safe Working Load	400 lbs. (181kg)
Overall Depth	26.5" (673mm)
Maximum Overall Height (raised)	70.75" (1746mm)
Minimum Overall Height (lowered)...	41.75" (1009mm)
External Width incl Rise Handles.....	23" (584mm)
Maximum Seat Height	30" (762mm)
Seat Width	14" (356mm)
Wheel Diameter.....	5" (127mm)
Unit Weight	78 lbs. (35kg)

Electrical Specifications

Standard 2-prong 110 VAC Power Cord (USA)
 Standard 2-prong 220 VAC Power Cord (International)
 Operating Environment: >+5°C <+40°C (41° to 104°F)
 Lithium-Ion Battery: 24 VDC

Maintain Privacy, Dignity, and Independence

These products are covered by one or more patents, including U.S. Patent Nos. 9,808,388 B2 & 10,835,434



Distributed By: HMS Medical Pty Ltd
 E: sales@hmsmedical.com.au

T: 1300 106 003 / 07 3117 9539
 www.hmsmedical.com.au

IndeeLift.com