



Instructions for Use

Cleaning:

Most cleaning agents utilized to clean OR Table pads are safe to use to clean our MicroArmor products. The coating has been specifically tested against the cleaning solutions below with no effect on the coating.

1. Rubbing Alcohol (70% isopropyl alcohol)
2. Bleach Solution (10% household-type bleach in cold, cool or warm water)
3. General Purpose soap or detergent type cleaners ("409", etc.) at their normal manufacturer recommended strength.
4. Germicidal wipes may also be used.

Wipe clean with wipes or cloths. **DO NOT SUBMERGE.**

Do not use any solutions heated to higher than 65 degrees Celcius.

DO NOT USE Tape and adhesive products as it can cause damage to the coating when being removed.

DO NOT ATTEMPT TO STERILIZE

AVOID Vigorous scrubbing, and cleaning with abrasive materials, scrubbers, sharp objects, etc.